# ODOFIN™

*Sniffin' Sticks*

DUFT-QUARTETT

**burgornt**  
MESSTECHNIK


DEUTSCH .....	4
ENGLISH.....	10
DANSK .....	15
NEDERLANDS .....	20
FRANÇAIS .....	26
ITALIANO .....	32
POLSKI .....	38
SVENSKA.....	44
ESPAÑOL.....	49
EESTI.....	55
SUOMI.....	60
EΛΛΗΝΙΚΑ .....	66
LATVIEŠU .....	72
PORTUGUÊS.....	77
SLOVENŠČINA .....	83
SLOVENČINA .....	88
ČESKY .....	93
MAGYAR.....	98
LEGEND.....	104

PZN (DQ):  
17635725

PZN (DQ2):  
17924961

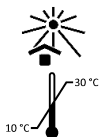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Uetersener Straße 6  
D - 25488 Holm  
☎ +49 4103 80076-0  
☎ + 49 4103 80076-29  
✉ info@burghart-mt.de  
🌐 www.burghart-mt.de

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**ENGLISH****Table of contents**

Intended Use .....	11
Content.....	11
User.....	11
Age group.....	11
Warning notices.....	11
Contraindication .....	12
Mechanism of action and area of application .....	13
Procedure of the training .....	13
Shelf Life .....	14
Storage.....	14
Disposal.....	14
Substances.....	14

**IMPORTANT**  
**READ CAREFULLY BEFORE USE**  
**KEEP FOR FUTURE USE**

## Intended Use

The "Duft-Quartett" is used to train the sense of smell in case of smell disorders. The "Duft-Quartett" is meant to enhance the sense of smell in many people or patients.

## Content

The "Duft-Quartett" consists of 4 felt tip sticks which are filled with 4 different odours.

The qualities for "Duft-Quartett" are "lemon" (stick 1), "eucalyptus" (stick 2), "clove" (stick 3) and "rose" (stick 4).

The qualities for "Duft-Quartett 2" are "menthol" (stick 1), "thyme" (stick 2), "mandarin" (stick 3) and "jasmine" (stick 4).

## User

The test is to be used exclusively by the owner. Children should only use the test under adult supervision.

## Age group

There is no age group restriction.

## Warning notices

All substances used in this test are not health hazardous in their used concentrations. Rarely allergic reactions show up after inhaling and skin contact. In case of allergic reaction contact the attending doctor and the manufacturer.

The user shall wear odour-free gloves during the procedure.

Use of the sticks only in an odourless or well-ventilated environment.

Only use the stick for recommended procedures and not for writing purposes.

Caution: People with allergies against odorants should not use the “Duft-Quartett” without consulting their treating physician beforehand.

Store the product safely and out of the reach of children.

If the tip of the Sniffin' Sticks accidentally comes in contact with the gloves, the gloves need to be replaced to avoid contamination of other odours.

Avoid any contact of the felt-tip with skin and especially mucosa (eyes, inner nose, mouth). In the case of touching any skin, rinse the area with water and mild soap. In case of touching the eyes, thoroughly rinse with water. If contact lenses are used, remove them and continue to rinse the eye with water. However, it must be disposed of because of the reduced shelf life of the odours and replaced by a new one.

### **Contraindication**

The odorants used in stick 1, 2 and 3 (in the “Duft-Quartett”) and in stick 2 and 3 (in the “Duft-Quartett 2”) can trigger an allergic reaction in rare cases. Please see a doctor immediately if an allergic reaction occurs and show this manual. The doctor is requested to inform the manufacturer immediately.

### Mechanism of action and area of application

Based on current knowledge the effect of the “Duft-Quartett” and “Duft-Quartett 2” are stimulation of the olfactory epithelium in the nose as well as enhancement of the sensation in the brain.

Continuous training with the “Duft-Quartett” or “Duft-Quartett 2” can reduce a dysfunction of the sense of smell, e.g. after a respiratory infection and can potentially be effective in case of other causes (e.g. craniocerebral injury).

The treatment requires continuous training with the "Duft-Quartett" over a longer period of time, following the instructions in the manual.

### Procedure of the training

Put on the provided cotton gloves. Hold the stick in one hand and open the cap of the stick with the other hand. Take care not to touch the tip of the stick with your skin or other objects. Hold the stick approx. 2 cm (0.8 in) beneath your nose. Inhale the odour intentionally for 15 seconds and put the cap back on afterwards. Please repeat the procedure for each odour. Per training session, you should do a total of 2 rounds with all 4 sticks, so that at the end you have smelled each stick for 30 seconds. Do this training in the morning and in the evening for 6 to 9 months. Usually, it takes from several weeks to a few months for the sense of smell to improve noticeably.

Hold the tip of the stick approx. 2 cm (0.8 inch) below the nose and centered between the nostrils.

Please observe the preceding contraindications and warnings when using the product.

### Shelf Life

The sticks are labelled with a lot-number and a best before date. Use beyond the expiry date may change the odorant quality and cause irritation.

### Storage

Refer to the symbols on the box how to store the test (dry, protect from sunlight). The temperature should be between 10 and 30°C.

**Caution:** Storing the test in the refrigerator can lead to a change in the intensity of the odorant.

### Disposal

Unusable sticks must be disposed of in their sealed condition in the household waste.

### Substances

Stick	Duft-Quartett	Duft-Quartett 2
1	(+)-Limonen	L-Menthol
2	Eucalyptol	Thyme scent
3	Eugenol	Mandarin scent
4	2-Phenylethanol	Jasmine aroma

The specific safety data sheets can be provided on request to the manufacturer.

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