# ODOFIN™

*Sniffin' Sticks*

Identification Test

**burgardt**  
MESSTECHNIK

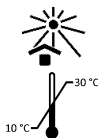
DEUTSCH .....	3
ENGLISH .....	15
DANSK .....	26
Legende/Legend .....	37

PZN (blue):  
17635731

PZN (purple):  
17850085

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Uetersener Straße 6  
 D - 25488 Holm  
 ☎ +49 4103 80076-0  
 ☎ + 49 4103 80076-29  
 ✉ info@burghart-mt.de  
 🌐 www.burghart-mt.de

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## Literatur

A Oleszkiewicz , V A Schriever , I Croy , A Hähner , Thomas Hummel

### **Updated Sniffin' Sticks normative data based on an extended sample of 9139 subjects**

Eur Arch Otorhinolaryngol. 2019 Mar;276(3):719-728. doi: 10.1007/s00405-018-5248-1. Epub 2018 Dec 15.

PMID: 3055435

A Sorokowska, E Albrecht, A Haehner, T Hummel

### **Extended version of the "Sniffin' Sticks" identification test: test-retest reliability and validity**

J Neurosci Methods. 2015 Mar 30; 243:111-4. doi: 10.1016/j.jneumeth.2015.01.034. Epub 2015 Feb 10.

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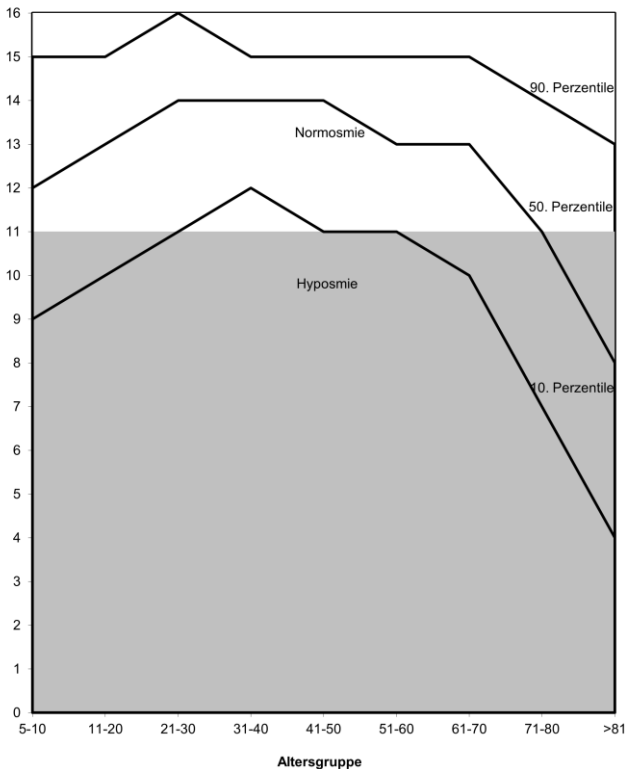
**IMPORTANT**  
**READ CAREFULLY BEFORE USE**  
**KEEP FOR FUTURE USE**

### Intended use

The identification test is intended to be used for the classification of the olfactory performance of humans. The test results can be used to differentiate between Normosmia and Hyposmia if the patient is cooperative. Please refer the chapter "Advanced smell tests" for warnings and additional information.

### User

The test is to be performed by healthcare professionals only.

### Age group

The test can be carried out starting at the age of 5 years. For children between 6 and 17 years of age, there is a special variant of the identification test (Kids Identification test LA-13-00407).

### Warnings

The olfactory sticks must not touch the patient's skin during the olfactory presentation. However, if the stick is inadvertently touched, it must be disposed of for hygienic reasons and because of the reduced shelf life of the odours and replaced by a new one.

All substances used in this test are not health hazardous in their used concentrations. Rarely allergic reactions show up after inhaling and skin contact. In case of allergic reaction contact the attending doctor and the manufacturer.

The user shall wear odour-free gloves during the procedure.

Use of the sticks only in an odourless or well-ventilated environment.

Only use the stick for recommended procedures and not for writing purposes.

Caution: Ask for intolerances before each test.

Store the product safely and out of the reach of patients and children.

If the tip of the Sniffin' Sticks accidentally comes in contact with the gloves, the gloves need to be replaced to avoid contamination of other odours.

### **Contraindication**

An allergy against one of the odorants used in the tests.

### **Patient information on the smell test**

The patient shall be informed in following manner:

"Your sense of smell shall be tested. This will be done with the "Sniffin' Sticks" which contain different odours. Please smell each stick. For each odour stick a multiple-choice card is provided. Please choose the term that matches your personal olfactory perception best. Due to the test method you need to choose, even if you smell nothing. In this case, please guess. Guessed answers are taken into account in the test evaluation. The full test takes about 15 minutes.

## Information for users

Here the ability to identify everyday smells by means of a card with 4 choices is determined. It is a multiple-forced-choice procedure, which means the patient must pick one of the 4 terms.

Altogether, 16 odours are presented in the Identification Test. The "Identification Test Plus 16" (purple) contains 16 additional smells. It serves as supplement to the Identification Test 16, blue, to avoid possible recognition of the sticks.

Use either the Identification Test (blue stick ends) or the Identification Test Plus 16 (purple stick ends). Primarily, both tests are available to make repeated tests of patients in a short interval (hours to days, e.g., before and after a surgery) possible, without the patient recalling the answers. If you want to use both tests together, use the average of both Identification Tests for evaluation.

The identification test may only be used in connection with the user manual, the holder and the "Sniffin' Sticks".

For refilling, only refill sets from the manufacturer are allowed to be used.

## Information for the Examiner

Patients often do not understand why they have to decide for a particular smell on the answer card, even though they did probably not smell anything. The patient may be given the additional explanation that this is a basic principle of modern test procedures and that accidentally correct answers are taken into account during evaluation of the test. During the evaluation it will be taken into account that usually a score greater than zero is reached even though the patient did not actually smell anything, but just hit the correct answer by chance. Thus, reaching this score indicates that the patient cannot smell or cannot properly smell.

## Requirement

The following criteria must be taken into account:

- To obtain the most reliable results possible, the patient should consume nothing other than water at least 30 minutes before the start of the test.
- Chewing gum, sweets or cigarettes must also be avoided.
- During the test, the patient is not to be given any indication of the correctness of the decision.
- The Identification test may only be used in combination with instructions for use, holders and sticks. Only the refill set supplied by the manufacturer may be used as a refill set.

## Test procedure

Before presenting the Sniffin' Stick, hand out the corresponding multiple-choice card. Pull the Sniffin' Stick out of the holder and remove the cap carefully. Sometimes a gentle rotation of the cap is necessary.

For testing, the opened odour stick is held 2 cm centred in front of both nostrils and the patient is asked to smell, e.g., using the sentence "Please sniff now". Each stick should be presented for 3-4 seconds. The interval between the different presentations should be around 30 seconds.

Ask the patient to choose a term on the choice card and mark the answer on the identification sheet. Be careful not to give the patient hints if the answer is correct or not.

16 sticks are presented to the patient one after another at an interval of approximately 30 seconds. Then enter the overall result on the evaluation sheet. Make sure to use the multiple-choice cards and the record sheet marked "blue ●" for the identification test (blue stick ends) and those marked "purple ▲" for the identification test Plus 16 (purple stick ends).

Please put the individual stick back into the block directly after use and with the cap closed. The sticks must be stored in an upright "overhead" position (white cap downwards). This prevents release of fragrance into the room.

### **Testing each side of the nose separately**

The olfactory performance of the left and right side of the nose can be tested individually. To perform this test, the opened stick is simply held in front of one nostril. The untested side is in this case closed by the patient by placing the pad of the right or left thumb on the nostril from below. The closure must not deform the nose. In this case the order of the presented sticks should be altered to reduce the predictability of the answer. It is e.g., possible to start with stick no. 1 at the left nostril and with stick no. 8 at the right nostril. Another option is the alternating application, on the left and right side, until all 16 smells have been presented on both sides. It is advisable to stay with the initially selected procedure and describe it on the enclosed evaluation sheet. Whichever method you choose, the important thing is to start identifying on the worse side.

### **Evaluation**

The answers of the patient marked on the identification sheet can easily be checked with the answer template.

The number of correct answers is counted and compared with normative data shown in the normogram on page 25. In this normogram the area for hyposmia is highlighted in gray. The

lines in the diagram show the smell ability correlated to age groups.

Since the Identification Test alone is not intended for a detailed assessment, the only possible statement that can be made is based on the norm values on page 25. If the result is within the range of the normosmia, there is no hint to a smell disorder of the patient.

### **Advanced smell test**

The identification test is a fast-screening test which was developed with the intent of saving time. The low numbers of different odours can be insufficient to differentiate normosmia and hyposmia, especially if the patient is not willing to cooperate. In cases like expert opinion for a lawsuit or unwilling patients an advanced smell test with the "Extended test" of the Sniffin' Sticks or olfactorily evoked potentials with an olfactometer should be performed.

### **Shelf Life**

The sticks are labelled with a lot-number and a best before date. Use beyond the expiry date may lead to incorrect results.

### **Storage**

Refer to the symbols for how to store the test (dry, protect from sunlight). The temperature should be between 10°C and 30°C.

The sticks must be stored in an upright "overhead" position (white cap downwards).

Caution:

Storage in the fridge can lead to a change in the intensity of the odour.

**Disposal**

Unusable sticks must be disposed of in their sealed state in the household waste.

**Ingredients**

Identification blue ●

Pen	Fragrance
No. 1	Orange oil
No. 2	Leather fragrance
No. 3	Cinnamaldehyde
No. 4	Peppermint oil
No. 5	Banana fragrance
No. 6	Citrus oil
No. 7	Anethole
No. 8	Turpentine oil
No. 9	Garlic oil
No. 10	Coffee oil
No. 11	Apple odour
No. 12	Clove oil
No. 13	Pineapple aroma odour
No. 14	Rose perfum oil
No. 15	Anethole
No. 16	Fish odour
Solvent	Diethyl phthalate
Solvent	Propylenglycolum

## Identification purple ▲

Pen	Fragrance
No. 1	Pear
No. 2	Cola
No. 3	Lilac
No. 4	Grapefruit
No. 5	Grass
No. 6	Raspberry
No. 7	Eucalyptus
No. 8	Ginger
No. 9	Coconut
No. 10	Lavender
No. 11	Melon
No. 12	Peach
No. 13	Mushroom aroma
No. 14	Smoked Meat
No. 15	Caramel
No. 16	Onion
Solvent	Diethyl phthalate
Solvent	Propylenglycolum

## Literature

A Oleszkiewicz , V A Schriever , I Croy , A Hähner , T Hummel  
**Updated Sniffin' Sticks normative data based on an extended sample of 9139 subjects**

Eur Arch Otorhinolaryngol. 2019 Mar;276(3):719-728. doi:  
10.1007/s00405-018-5248-1. Epub 2018 Dec 15.

PMID: 3055435

A Sorokowska, E Albrecht, A Haehner, T Hummel

**Extended version of the "Sniffin' Sticks" identification test:  
test-retest reliability and validity**

J Neurosci Methods. 2015 Mar 30;243:111-4. doi:  
10.1016/j.jneumeth.2015.01.034. Epub 2015 Feb 10.

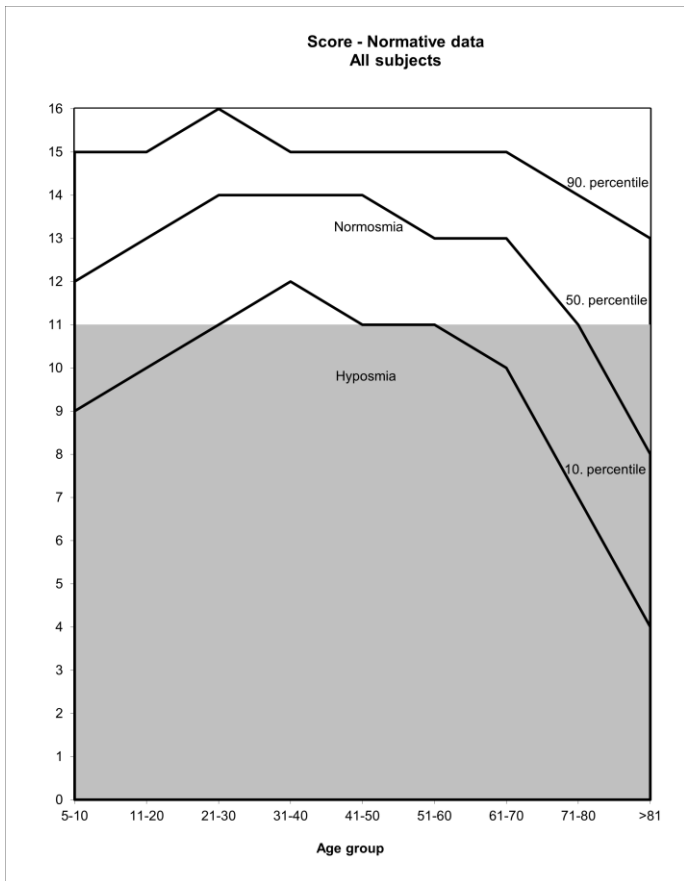
PMID: 25677404

## Normative data

The following graph can be used for the evaluation. The graph was created with the data from the norm data study mentioned under "Literature". The graph is not sex-specific, as the difference of the olfactory performance between the sexes is not of a relevant magnitude for the evaluation.

The classification of normosmia and hyposmia is based on the data of the best-performing age group (21-30) regarding the ability to smell. The distinction from anosmia can only be determined with the Extended Test.

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