# ODOFIN™

*Sniffin' Sticks*

## Kids Identification Test

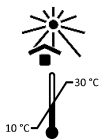
# ODOFIN™ *Sniffin' Sticks* Kids Ident. Test

DEUTSCH	3
ENGLISH.....	12
LEGENDE/LEGEND.....	22

PZN: 17850122

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Uetersener Straße 6  
D - 25488 Holm  
☎ +49 4103 80076-0  
☎ + 49 4103 80076-29  
✉ info@burghart-mt.de  
🌐 www.burghart-mt.de

## DEUTSCH

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**IMPORTANT  
READ CAREFULLY BEFORE USE  
KEEP FOR FUTURE USE**

### **Intended use**

The "Kids Ident. Test" is used for the orientational classification of children's ability to smell.

### **User**

The test is to be performed by healthcare professionals only.

### **Age group**

The test is intended for children aged 6 to 17 years.

### **Warnings**

The olfactory sticks must not touch the patient's skin during the olfactory presentation. However, if the stick is inadvertently touched, it must be disposed of for hygienic reasons and because of the reduced shelf life of the odours and replaced by a new one.

All substances used in this test are not health hazardous in their used concentrations. Rarely allergic reactions show up after inhaling and skin contact. In case of allergic reaction contact the attending doctor and the manufacturer.

The user shall wear odour-free gloves during the procedure.

Use of the sticks only in an odourless or well-ventilated environment.

Only use the stick for recommended procedures and not for writing purposes.

Caution: Ask for intolerances before each test.

Store the product safely and out of the reach of patients and children.

If the tip of the Sniffin' Sticks accidentally comes in contact with the gloves, the gloves need to be replaced to avoid contamination of other odours.

### **Contraindication**

An allergy against one of the odorants used in the test.

### **Patient information on the smell test**

The patient or the guardian may be informed in the following manner: "Your sense of smell shall be tested. This will be done with the "Sniffin' Sticks" which contain different odours. Please smell each stick. For each odour stick a choice card with 4 choices is provided. Please choose the picture that matches your personal olfactory perception best. Due to the test method you need to pick a choice, even if you smell nothing. guessed answers are taken into account in the test evaluation. The full test takes about 15 minutes.

### **Information for users**

The test method of the "Kids Ident. Test" is based on the "multiple-forced-choice" method. This means that the patient must answer after each Sniffin' Stick according to the presented choice card, even if the patient detects nothing. This procedure does not lead to a corruption of the score because only a random value can be achieved. This value is considered in the test evaluation. To apply this method a patient's answer for each Sniffin' Stick is mandatory otherwise the test can not

be evaluated. If the test results do not reflect the patient's stated ailment, it is recommended to perform advanced smell tests (please refer chapter "Advanced smell tests"). The "Kids Ident. Test" may only be used in connection with the user manual, the holder and the Odofin sniffin' sticks. For refilling, only refill sets from the manufacturer are allowed.

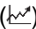
### Test procedure

Please make sure that the patient does not consume food, drinks or cigarettes 30 minutes before testing (only exception is the drinking of tap water). Before presenting the Sniffin' Stick, hand out the corresponding choice card. Pull the Sniffin' Stick out of the holder and remove the cap carefully. Sometimes a gentle rotation of the cap is necessary.

For testing, the opened odour stick is held 2 cm centred in front of both nostrils and the patient is asked to smell, e.g. using the sentence "Please sniff now". Each stick should be presented for 3-4 seconds. The interval between the different presentations should be around 30 seconds.

Ask the patient to choose a picture on the choice card and mark the answer on the identification sheet. Be careful not to give the patient hints if the answer is correct or not. Repeat this procedure for each stick with a minimum pause of 30 seconds. You then fill in the overall result on the evaluation sheet (). For odour testing, only the cap of one stick should be removed. The cap must be put back on the odour stick immediately after the odour presentation. The odour stick must then be immediately placed back into the holder with the cap facing down. The test results can be used to differentiate between

normosmia and a reduced smell ability. The Kids Identification Test is not suitable for detecting a loss of the ability to smell (anosmia). Please refer to the chapter "Advanced smell tests" for additional information.

### **Separate testing each side of the nose**

If the test is to be carried out left or right sided, the opened stick is simply held in front of one nostril. The untested side is in this case closed by the patient by placing the pad of the right or left thumb on the nostril from below. The closure must not deform the nose. In this case the order of the presented sticks should be altered to reduce the predictability of the answer. It is e.g., possible to start with stick no. 1 at the left nostril and with stick no. 8 at the right nostril. Another option is the alternating application, on the left and right side, until all 16 smells have been presented on both sides. It is advisable to stay with the initially selected procedure and describe it on the enclosed evaluation sheet. Whichever method you choose, the important thing is to start identifying on the worse side.

### **Evaluation**

The answers noted on the identification sheet can be easily checked with the help of the answer template (⊕) after the patient has completed the test. The number of correct answers is added up and compared with the normative values on page 21. The 10th percentile was taken as the cut off between normosmia and reduced olfactory function.

### **Advanced smell test**

The "Kids Ident. Test" is a fast-screening test which was developed with the intent of time saving. The low numbers of different odours can be insufficient to differentiate normosmia, hyposmia and anosmia, especially if the patient is not willing to

cooperate. In these cases, an advanced smell test with the "Extended test" of the Sniffin' Sticks or olfactorily evoked potentials with an olfactometer should be performed.

### **Shelf Life**

The sticks are labelled with a lot-number and a best before date. Use beyond the expiry date may lead to incorrect results.

### **Storage**

Refer to the symbols (see page 22) how to store the test (dry, protected from sunlight). The temperature should be between 10 and 30°C.

The sticks must be stored in an upright "overhead" position.

Caution: Storing the test in the refrigerator can lead to a change in the intensity of the fragrance.

### **Disposal**

Unusable sticks must be disposed of in their sealed condition in the household waste.

### Accessories and evaluation material

Accessories and evaluation material	Item number
Holder	LA-13-00339
Instruction for use	LA-18-00098
Answer template	LA-18-00099
Evaluation and Identification block	LA-18-00133
Multiple-Choice-Cards	LA-18-00100

### Used Substances

Stick	Chemical
No. 1	Apple odour
No. 2	Banana fragrance
No. 3	Butter aroma
No. 4	Coffee oil
No. 5	Grass Cis-3-hexen-1-ol
No. 6	Fish odour
No. 7	Rose perfum oil
No. 8	Citrus oil
No. 9	Onion aroma
No. 10	Orange oil
No. 11	Peach aroma
No. 12	Strawberry aroma
Solvent	Diethylphthalate
Solvent	Propylenglycolum

The specific safety data sheets can be provided on request to the manufacturer.

## Literature

Janine Gellrich, Lisa-Marie Sparing-Paschke, Theresa Thieme, Katharina Schwabe, Annika Dworschak, Thomas Hummel, Valentin A Schriever

### **Normative data for olfactory threshold and odor identification in children and adolescents**

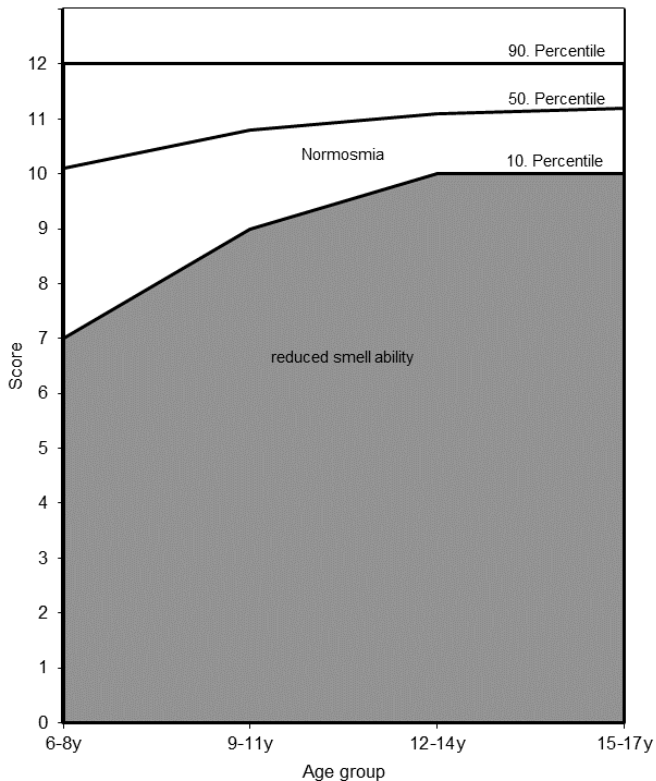
Int J Pediatr Otorhinolaryngol. 2019 Aug;123:5-9. doi: 10.1016/j.ijporl.2019.01.009. Epub 2019 Jan 9.

PMID: 31054538

### **Notice regarding the percentiles**

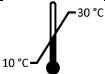
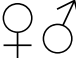
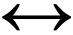
The percentiles shown in the normogram are values for healthy subjects.

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