ODOFIN™

Sniffin' Sticks

Screening 12 Test

burgardt
MESSTECHNIK

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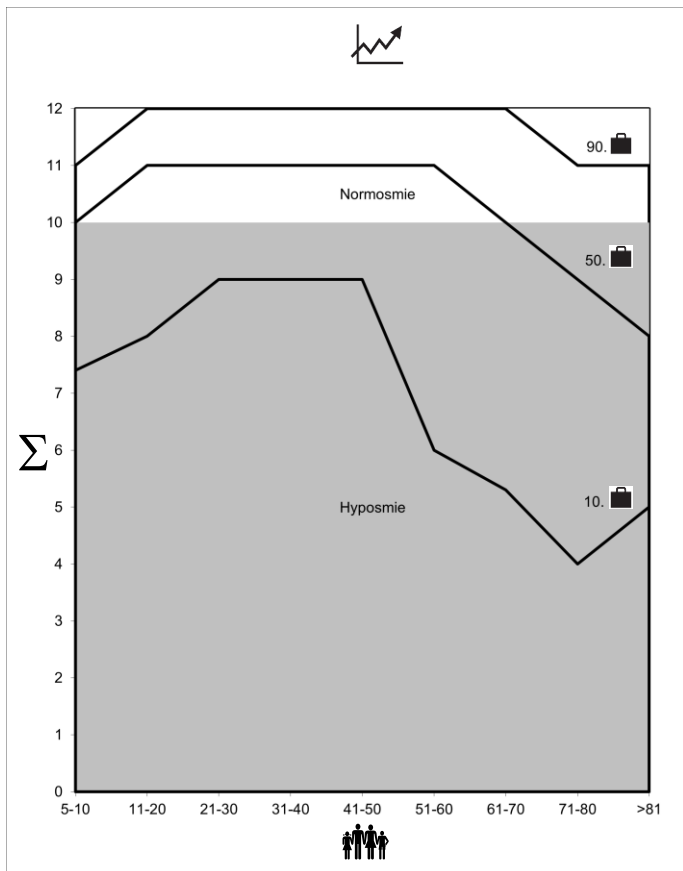
IMPORTANT
READ CAREFULLY BEFORE USE
KEEP FOR FUTURE USE

Intended use

The Screening 12 test is used for the orientational classification of the patient's ability to smell.

User

The test is to be performed by healthcare professionals only.

Age group

The test can be carried out starting at the age of 5 years. There is a special variant for children between 6 and 17 years of age of the Screening 12 test (Kids identification test LA-13-00407).

Warnings

The olfactory sticks must not touch the patient's skin during the olfactory presentation. However, if the stick is inadvertently touched, it must be disposed of for hygienic reasons and because of the reduced shelf life of the odours and replaced by a new one.

All substances used in this test are not health hazardous in their used concentrations. Rarely allergic reactions show up after inhaling and skin contact. In case of allergic reaction contact the attending doctor and the manufacturer.

The user shall wear odour-free gloves during the procedure.

Use of the sticks only in an odourless or well-ventilated environment.

Only use the stick for recommended procedures and not for writing purposes.

Caution: Ask for intolerances before each test.

Store the product safely and out of the reach of patients and children.

If the tip of the Sniffin' Sticks accidentally comes in contact with the gloves, the gloves need to be replaced to avoid contamination of other odours.

Contraindication

An allergy against one of the odorants used in the test.

Patient information on the smell test

The patient shall be informed in following manner:

"Your sense of smell shall be tested. This will be done with the "Sniffin' Sticks" which contain different odours. Please smell each stick. For each odor stick a choice card with 4 choices are provided. Please choose the term that matches your personal olfactory perception best. Due to the test method you need to pick a choice, even if you smell nothing. In this case, please guess. The score is a sum of all correct answers. The evaluation of this test considers eventual correct answers which are generated by accident. Guessed answers are not corrupting the test evaluation. Please do not touch the tip of the stick, or it would become unusable and can lead to irritation of the skin and invalidation of the test."

The full test takes about 15 minutes.

Information for users

The principle underlying the Screening 12 test is based on the "multiple-forced-choice" method. This means that the patient must answer after each Sniffin' Stick according to the presented choice card, even if the patient detects nothing. This procedure does not lead to a corruption of the score because only a statistic random value of guessing can be achieved. This value is considered in the test evaluation. To apply this method a patient's answer for each Sniffin' Stick is mandatory otherwise the test can not be evaluated. If the test results do not reflect the patient's stated ailment, it is recommended to perform advanced smell tests (please refer chapter "Advanced smell tests").

The Screening 12 test may only be used in combination with the instructions for use, holder and sticks.

Only the refill set from the manufacturer may be used as a refill set.

Test procedure

Please make sure that the patient does not consume food, drinks or cigarettes 30 minutes before testing (only exception is the drinking of tap water). Before presenting the Sniffin' Stick, hand out the corresponding choice card. Pull the Sniffin' Stick out of the holder and remove the cap carefully. Sometimes a gentle rotation of the cap is necessary.

For testing, the opened odour stick is held 2 cm centred in front of both nostrils and the patient is asked to smell, e.g. using the sentence "Please sniff now". Each stick should be presented for 3-4 seconds. The interval between the different presentations should be around 30 seconds.

Ask the patient to choose a term on the choice card and mark the answer on the identification sheet. Be careful not to give the patient hints if the answer is correct or not. Repeat this procedure for each stick with a minimum pause of 30 seconds. You then fill in the overall result on the evaluation sheet () . For odour testing, only the cap of one stick has to be removed. The cap must be put back on the odour stick immediately after the odour presentation. The odour stick must then be immediately placed back into the holder with the cap facing down. The test results can be used to differentiate between normosmia and a reduced smell ability (hyposmia). The Screening 12 test is not suitable for detecting a loss of the ability to smell (anosmia). Please refer to the chapter "Advanced smell tests" for additional information.

Separate testing each side of the nose

If the test is to be carried out left or right sided, the opened stick is simply held in front of one nostril. The untested side is in this case closed by the patient by placing the pad of the right or left thumb on the nostril from below. The closure must not deform the nose. In this case the order of the presented sticks should be altered to reduce the predictability of the answer. It is e.g., possible to start with stick no. 1 at the left nostril and with stick no. 8 at the right nostril. Another option is the alternating application, on the left and right side, until all 16 smells have been presented on both sides. It is advisable to stay with the initially selected procedure and describe it on the enclosed protocol sheet. Whichever method you choose, the important thing is to start identifying on the worse side.

Evaluation

The answers noted on the identification template can be easily checked with the help of the answer template (🗑️) after the test person has completed the test. The number of correct answers is added up and compared with the normative values on page 23. In this normogram, the area of hyposmia is marked in grey. The lines indicate the smelling ability as a function of age.

Advanced smell test

This olfactory test is a screening procedure that requires little time. Due to the small number of olfactory samples, there may occasionally be difficulties at the limits in differentiating between normal and impaired olfactory abilities. This is particularly true in the case of reduced cooperation on the part of the test person or in the case an expert opinion is needed. In these cases, either the "Extended Test of Sniffin' Sticks" or the recording of olfactory evoked potentials should be used.

Shelf Life

The sticks are labelled with a lot-number and a best before date. Use beyond the expiry date may lead to incorrect results.

Storage

Refer to the symbols (see page 33) for how to store the test (dry, protected from sunlight). The temperature should be between 10°C and 30°C.

The sticks must be stored in an upright "overhead" position.

Caution: Storage in the fridge can lead to a change in the intensity of the odour.

Disposal

Unusable sticks must be disposed of in their sealed condition in the household waste.

Substances used

Stick	Chemical
No. 1	Orange oil
No. 2	Leather fragrance
No. 3	Cinnamaldehyde
No. 4	Peppermint oil
No. 5	Banana fragrance
No. 6	Citrus oil
No. 7	Anethole
No. 8	Coffee oil
No. 9	Clove oil
No. 10	Pineapple aroma odour
No. 11	Rose perfum oil
No. 12	Fish odour
Solvent	Diethylphthalate
Solvent	Propylenglycolum

The specific safety data sheets can be obtained on request from the manufacturer.

Literature

T Hummel, C G Konnerth, K Rosenheim, G Kobal

Screening of olfactory function with a four-minute odor identification test: reliability, normative data, and investigations in patients with olfactory loss

Ann Otol Rhinol Laryngol. 2001 Oct;110(10):976-81.doi: 10.1177/000348940111001015.

PMID: 11642433

Čičelienė J, Vaičys Ž, Rastenytė D.

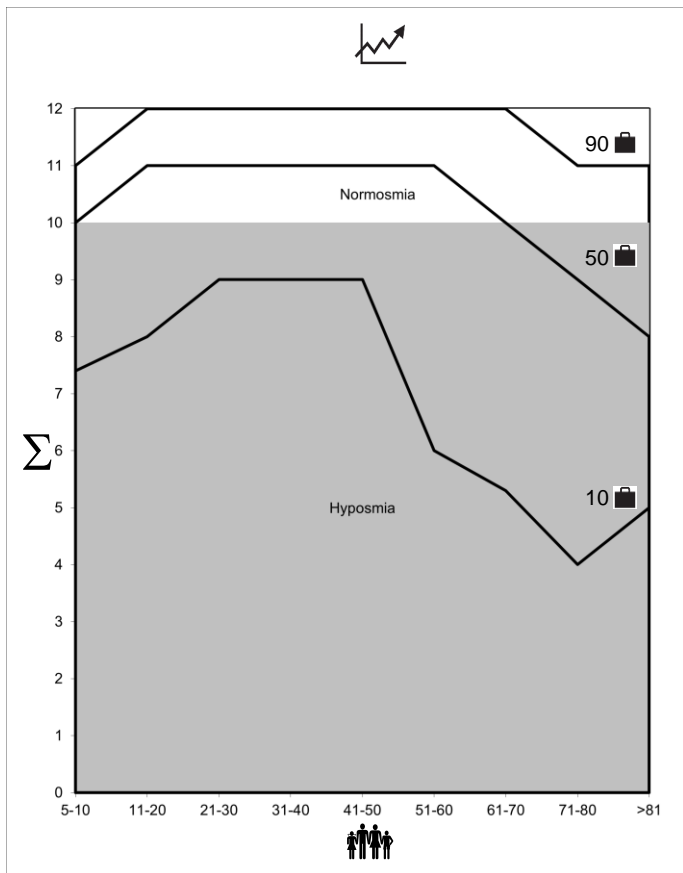
Development of the Lithuanian Version of Sniffin' Sticks 12 Odor Identification Test.

Medicina (Kaunas). 2018 Apr 11;54(2):13. doi: 10.3390/medicina54020013.

PMID: 30344244; PMCID: PMC6037244.

Normative Values

The following graph can be used for the evaluation. The graph was created with the data from the norm data study mentioned under "Literature". The graph is not sex-specific, as the differences between sexes are below the resolution of the test.

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